RESTA

BPA350 AOJ-300

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English

We sincerely thank you for your purchase of this CRESTA medical product.

To ensure optimal use and complete satisfaction, we recommend that you carefully read the user manual before using the product. This applies even if you are already familiar with similar products

This medical device is intended to supplement, not replace, the prescriptions and/or instructions of your doctor or nurse.

If you have any questions or concerns, always consult your doctor or healthcare professional.

For manuals and update information for this product, please visit the product page on our website: www.cresta-care.com.

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Thank you for purchasing this Arm Blood Pressure Monitor. The monitor can be used and store measurement results for two users. It uses the oscillometric method of blood pressure measurement. This means this monitor detects your blood movement through your brachial artery and converts the movements into a digital reading.

The device can be used in homecare environment, and the patient is an intended operator, and all the functions can be safely used.

1. Unpacking Inspection

Before use, please open the package carefully and check whether all the parts are available according to the following packing list and whether the parts are damaged during transportation, and then install and operate in strict accordance with the manual.

2. Packing List

No.	Name	Quantity
1	Arm Blood Pressure Monitor	1
2	Cuff 22-42 cm	1
3	Instruction Manual	1
4	Pouch	1
5	Quick start guide	1

3. Safety Precautions

The warnings and illustrations shown in the manual are intended to enable you to use the product safely and correctly, thus preventing harm to you and others, specific meanings of which are shown as follows:

Legend, mark and meaning	
	Warning information, refer to the attached document
	BF-type anti –shock degree for the application part
	Comply with local regulations
	Consult the instructions for use
	Keep dry
	Low voltage prompt
	Keep out of the sun
	Vertical upward
IP21	The device is protected against splashing water. Water splashed against the enclosure from any direction shall have no harmful effects.
RoHS	RoHS mark
CE ₀₁₂₃	CE mark

	Manufacturer
	Date of manufacture
	Serial number
	Lot number
	EU authorized representative

4. Product Composition

This product is composed of the main body and cuff.

5. Intended Use / Instructions for Use

The Arm Blood Pressure Monitor is intended to measure the systolic pressure and diastolic pressure, as well as the pulse rate of adult person via non-invasive oscillometric technique at medical facilities or at home.


6. Contraindication

No known contraindication existing.


RESTA

7. Product Parts

(1) Main body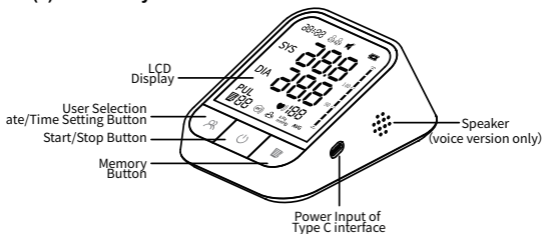

(2) Display screen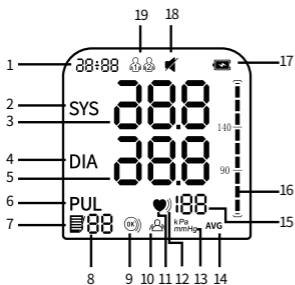

1. Date and Time
2. Systolic Blood Pressure Icon
3. Systolic Blood Pressure Value
4. Diastolic Blood Pressure Icon
5. Diastolic Blood Pressure Value
6. Pulse Display
7. Memory Icon
8. Memory Number
9. Detection for Cuff Wrapping Icon
10. Indication for Wrong Operation
11. Heartbeat Icon
12. Irregular Heartbeat Icon
13. Unit of Pressure
14. Average Value Icon
15. Pulse Rate Value
16. Blood Pressure Classification
17. Low Battery Icon
18. Voice On/Off Icon
19. User Icon


8. Preparation

(1) Install the battery

1) Open the battery cover according to the method shown in the figure.

2) Place 4 AAA batteries in the battery compartment, and pay attention to the electrode indication of the batteries.

Install the battery as indicated in the picture right under this sentence.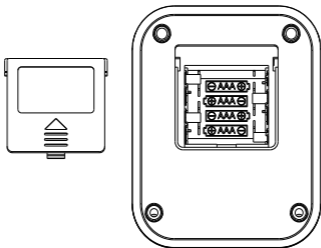

(2) Battery replacement

 If the product won't be used for a long time (over 3 months), please take out the batteries.


(3) Type C connection for power supply (The cable is optional)

The product power can be supplied by plugging into D.C. 5V external power supply through a Type C interface.

9. Function Setting

(1) User mode

In standby mode, press the "👤" button to enter the user group selection interface. Then press the "👤" button again to switch and select user groups.

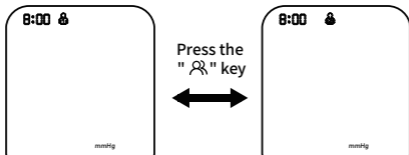
(2) Year/month/date setting

In standby mode, press "👤" for about 3 seconds to enter the date setting, and "year" will flash. Press "📄" to adjust to the desired year, and then press "👤" to confirm the selection. When the "year" is set, it will automatically enter the month setting. At this time, the "month" icon will flash. You can switch to the desired value by pressing the "📄" button. Follow the same step to set "date", "hour", and "minute".

(3) Voice on/off setting (for voice version only, optional function)

1) After the time setting is finished, it will automatically enter the voice setting (the screen will display "SP"). Press "📄" button to turn on or turn off the sound, "ON" means turn on the sound, "OFF" means turn off the sound, press "👤" button to confirm the selection.

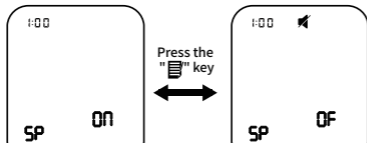2) Voice broadcast content:

Attention, please!

Please keep silent and relax.

Keep the cuff at the heart level.

① If your blood pressure is in normal range:

Thank you.

Your blood pressure is:

Systolic pressure xxx millimeter of mercury pillar.

Diastolic pressure xx millimeter of mercury pillar.

Your pulse is xx beats per minute.

Measuring result is normal.

Thank you.

Wish you a good health.

② If your blood pressure is not in normal range :

Measuring result is abnormal.

Your blood pressure is high/low.

Could not measure your pulse at this time.

③ Measuring error :

Error,(Mistake), please measure again

④ Other operation :

Average Blood Pressure


Last time your blood pressure was xxx, your pulse was xx

Low battery, please replace the battery

Low battery, please charge in time

(4) Unit display setting

In standby mode, keep pressing "  " button for about 10 seconds to enter the unit selection, and press "  " to switch mmHg/kPa. Press "  " to confirm the selection; The default unit is mmHg.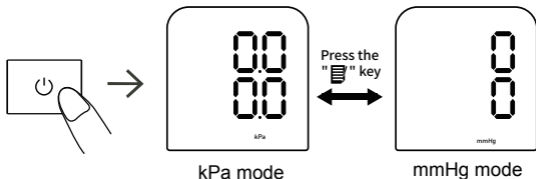


10. How to Take Proper Measurements

(1) Preparation before measurement


- Take off the clothes on the arm.
- Always measure in the same arm (generally the left arm).
- Remain still and keep quiet during measurement.
- Relax as much as possible and do not talk during measurement.
- Measure your blood pressure at about the same time every day.
- Do not measure right after physical exercise or a bath. Rest for 20 to 30 minutes before taking the measurement.
- Readings under the conditions listed below may affect results:
Within an hour after dinner, after having wine, coffee, tea, sports; talking, being nervous, being in unsteady mood, bending forward, moving, room temperature dramatically changing during measuring; inside a moving vehicle, repeated and continuous measuring.

(2) Applying the arm cuff

1) Connect the arm cuff to your monitor by inserting the air plug into the air jack securely.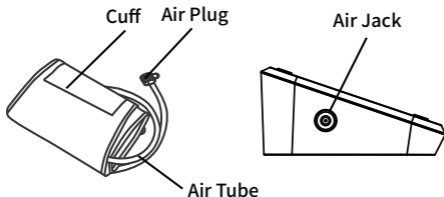


2) Place your hand through the cuff loop. Pull the cuff until it reaches your upper left arm.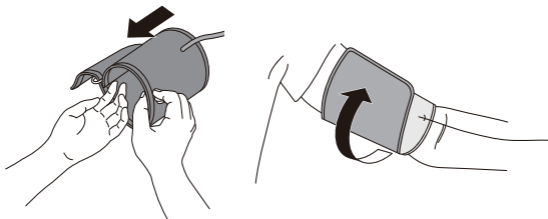

Note

- The bottom edge of the arm cuff should be 0.79-1.18 inches (2 to 3 cm) above the inside elbow. The air tube is on the inside of your arm and aligned with your middle finger.

RESTA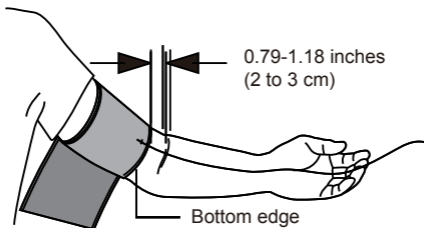

- Make sure that the air tube is positioned on the inside of your arm and wrap the cuff securely, so it can not move around your arm.

Note: Repeated measurement will result in blood congestion in the arm, which will affect the measurement result.

How to avoid blood congestion and ensure the repeated measurement is accurate?

You can raise the left hand and hold the fist several times, or take off the cuff and rest for at least 2-3 minutes before taking the measurement.

(3) Sitting correctly

To take a measurement, you need to be relaxed and comfortably seated in a room with a comfortable temperature. Place your arm on the table.

RESTA

- Sit in a comfortable chair with your back and arm supported.
- Keep your feet flat and your legs uncrossed.
- The cuff should be placed on your arm at the same level as your heart, with the arm resting comfortably on a table.


(4) Taking a measurement

Start measurement after wearing the cuff:

Press the "⏻" button and the monitor will begin to inflate.

Please do not move or talk during the measurement.

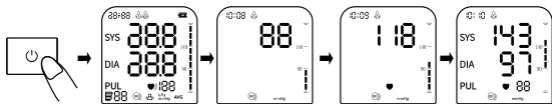Note: If you feel uncomfortable during the measurement, press the "⏻" button immediately to stop measure. When the air pressure is filled to a certain value, the value on the display screen will slowly drop at a certain speed, and the heartbeat symbol will flash. After the measurement is completed, the systolic pressure, diastolic pressure, and pulse measurements will be displayed on the screen.

Note: Consult your doctor if unexpected readings are obtained.

(5) Memory function



1) This monitor stores 99 sets of memories of two users. After each measurement, the monitor will automatically store the measurement data. When the memory is full, the old measurement data will be overfilled with the new one.

2) In standby mode, press the "" button once and the machine will display the average value of the blood pressure measured for the first three times. Press the "" button again, the first memory will be displayed. Press the "" button again and the rest memories will be displayed one by one.

(6) Delete memory

Keep pressing the "" button for 3 seconds to delete measurements stored on the device and the "" icon will appear on the screen.

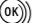(7) Self check for cuff strap

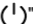The "" icon is always displayed on the screen when the cuff is worn correctly. When the cuff is worn too loosely, the "" icon will always flash to remind you. If the "" icon is flashing all the time, please press the "" button to stop the measurement.

(8) "Keep Still" indication

The "" icon flashes when the user moves the body or

shakes the arm during the measurement, which may cause incorrect measurement results. Please measure again.

11. Contraindications, Precautions, Warnings and Prompt Instructions

- No maintenance or servicing when using.
- The patient is an intended operator, and all the functions can be safely used.
- Dispose of the monitor when its service life is reached.

Follow local regulations regarding the disposal of such product.

- When the performance changes (such as: inaccurate measurement or abnormal display), please stop using it immediately and contact the after-sales service personnel in time.
- Do not leave the small parts where children can reach them.

Children may swallow them. If a child accidentally swallows them, battery cover, please contact a doctor immediately.

- Maintenance should be done by the operator as suggested
- Portable RF communications equipment (including peripherals such

as antenna cables and external antennas)

should be used no closer than 30cm (12inches) to any part of the this device, including cables specified by the manufacturer. Otherwise, degradation of the performance of

this equipment could result.

- When the ambient temperature is less than 5°C, please take the device to the place where the ambient temperature is between 5°C~40°C at least 1 hour; When the ambient temperature is higher than 40°C, please take the device to the place where the ambient temperature is between 5°C~40°C at least 2 hours.
- DO NOT use this monitor on infants, toddlers, children or persons who cannot express themselves.
- DO NOT adjust medication based on readings from this blood pressure monitor. Take medication as prescribed by your physician. ONLY a physician is qualified to diagnose and treat high blood pressure.
- DO NOT use this monitor on an injured arm or an arm under medical treatment.
- DO NOT apply the arm cuff on your arm while on an intravenous drip or blood transfusion.
- DO NOT use this monitor in areas containing high frequency (HF) surgical equipment, magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners. This may result in incorrect operation of the monitor and/or cause an inaccurate reading.
- DO NOT use this monitor in oxygen rich environments or near flammable gas.

RESTA

- Consult with your physician before using this monitor if you have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation; arterial sclerosis; poor perfusion; diabetes; pregnancy; pre-eclampsia or renal disease. NOTE that any of these conditions in addition to patient motion, trembling, or shivering may affect the measurement reading.
- NEVER diagnose or treat yourself based on your readings. ALWAYS consult with your physician.
- To help avoid strangulation, keep the air tube and type C cable away from infants, toddlers and children.
- Stop using this monitor and consult with your physician if you experience skin irritation or discomfort.
- Consult with your physician before using this monitor on an arm where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present because of temporary interference to blood flow which could result in injury.
- Consult with your physician before using this monitor if you have had a mastectomy.
- Consult with your physician before using this monitor if you have severe blood flow problems or blood disorders as cuff inflation can cause bruising.
- DO NOT take measurements more often than necessary because bruising, due to blood flow interference, may occur.

RESTA

- ONLY inflate the arm cuff when it is applied on your upper arm.
- Remove the arm cuff if it does not start deflating during a measurement.
- DO NOT use this monitor for any purpose other than measuring blood pressure.
- DO NOT disassemble or attempt to repair this monitor or other components. This may cause an inaccurate reading.
- DO NOT use in a location where there is moisture or a risk of water splashing this monitor. This may damage this monitor.
- DO NOT use this monitor in a moving vehicle such as in a car.
- DO NOT drop or subject this monitor to strong shocks or vibrations.
- DO NOT use this monitor in places with high or low humidity or high or low temperatures.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic

emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

12. Common Q & A on Blood Pressure

Q1: Why is the blood pressure value obtained at home lower than that obtained at the hospital?

- The blood pressure difference between home and hospital measurements is about 20 mmHg - 30 mmHg (2.7 kPa - 4.0 kPa). This is because individuals tend to be more relaxed at home than at the hospital.
- In addition, when the device is placed at a position over the heart, the blood pressure value tends to be much lower than it actually is. Ensure the device is positioned right at the heart level.

Q2: Why is the blood pressure value obtained at home higher than that obtained at the hospital?

- The anti-hypertensive drug might have lost its efficacy. Kindly adhere to your doctor's instructions.
- The cuff might not be in the correct position. If the cuff is not placed right, no arterial pressure value will be obtained, and the blood pressure value might be much higher than it is. Therefore, properly position the cuff.
- The cuff is not tight enough. If the cuff is loose, the compression force might fail to transmit to the artery,

causing the blood pressure value to be much higher than it is. Therefore, re-adjust and tighten the cuff further.

- The patient is not sitting correctly during the measurement. Slouching, tilting, bending, and sitting cross-legged are not encouraged while taking blood pressure measurements due to increased abdominal pressure or the arm position being below the heart. Kindly take readings in the correct posture.

Q3: When can I obtain better measurements?

- Measurements are best taken in the mornings right after you urinate or when your mind and body are stable. We recommend taking readings at the same time of the day, every time.

13. Abnormal Phenomena and Handling

* When the measurement is abnormal, the following symbols will appear. Please use the correct method for measurement.

LCD display information	Remark
Er U	The inflation can not reach 30 mmHg in 12 seconds.
Er H	The inflation reaches 295mmHg, it deflates automatically after 20ms.
Er 1	The pulse rate is not detected correctly.
Er 2	Too much disturbance (Move, talk, or magnetic disturbance during a measurement.)
Er 3	The test result is abnormal.
Er 23	SYS value reads lower than 57mmHg
Er 24	SYS value reads hinger than 255mmHg
Er 25	DIA value reads lower than 25mmHg
Er 26	DIA value reads hinger than 195mmHg

* Troubleshooting

RESTA

Anomaly	Possible Faulty	Solution
Failure to power on	Whether the power is insufficient	Replace the batteries or insert the Type-C charging cable for power supply
	Whether the positive and negative poles of the battery are installed reversely	Install the batteries correctly
No pressurizing	Whether the air tube plug is inserted tightly	Insert the air tube plug firmly into the jack.
	Whether the air tube is broken or leaked	Please contact the dealer to replace with a new cuff.
Unable to measure due to the display error	Whether the arm is moved when pressurization	Keep your arm and body still.
	Whether you talk during the measurement	Keep quiet while measuring the blood pressure.
Air leakage of the cuff	Whether the cuff is twined too loose	Please tighten the cuff
	The airbag of the cuff is ripped	Please contact the dealer to replace with a new cuff.

If the blood pressure still cannot be measured after trying the above-stated solutions, please contact the dealer. Do NOT attempt to disassemble the device by yourself.

14. Cleaning and Disinfection

14.1 Cleaning

- 1) Remove the batteries before cleaning.
- 2) The body of monitor can be cleaned with a clean soft cloth.
- 3) The body can be wiped with a clean soft cloth stained with a small amount of neutral detergent or water.

Do not use any corrosive cleaning agent. When cleaning, be careful not to immerse any part of the monitor to avoid liquid flow into the instrument.

14.2 Disinfection

Recommended disinfecting agent:

Isopropanol solution with 70% concentration

Disinfection steps:

- 1) Wipe the machine body with a clean soft cloth stained with a small amount of the above disinfectant, and dry it immediately.
- 2) The body of the monitor can be wiped with a clean cloth stained with a small amount of 75% medical alcohol for disinfection.

Do not disinfect by such method as high temperature steam or ultraviolet irradiation, which might damage the instrument or accelerate the aging!

It is suggested to disinfect the monitor before and after use each time. Each time of disinfection shall be completed within 1min. The number of repeated disinfection each time shall not exceed 2 times.

14.3 Disposal

Dispose of your monitor, other components and optional accessories according to applicable local regulations. Unlawful disposal may cause environmental pollution.

Notes

- Do not bend or crease the air tube excessively.
- Do not store your monitor and other components:
- If your monitor and other components are wet.
- In locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapors such as bleach.
- In locations exposed to vibrations or shocks.

15. Upkeep and Maintenance

water or neutral detergent

- Always keep the surface of monitor clean and tidy, helpful to prolong the service life of Blood Pressure Monitor.
- If the host is dirty, please wipe with a dry soft cloth. If the dirt cannot be eliminated easily, wipe with a soft cloth stained with water or neutral detergent, and then dry with a dry cloth.
- No maintenance or maintenance required when using the device.

Do not allow water or other liquids to flow into the host.

16. Specifications

Display mode	Digital LCD display
Measuring method	Oscillometric Measurement
Measuring part	Upper arm
Pneumatic pressure measuring range	0~295 mmHg
Overpressure	300 mmHg

RESTA

protection limit			
Measuring range	Pressure value	SYS: 57~255 mmHg; DIA: 25~195 mmHg;	
	Pulse rate	40~199 bpm	
Accuracy	Pressure value	±3 mmHg	
	Pulse rate	±5%	
Low battery	4.2V±0.1V: low battery; < 4.0V±0.1V: item will be turned off		
Anti Electronic Shock Type	Internal Power Supply		
Auto power-off	1 minute without operation		
Power source	4*AAA batteries or d.c. 5V Type-C charging cable		
Anti Electronic Shock Degree	Type BF		
Operation mode	Continuous operation		
Protection against harmful ingress of water or particular matter	IP21		
weight	About 316g(without batteries)		
Dimension	108mm(length)*139mm(width)* 62mm(height)		
Screen size	81.4mm(length)* 82.9mm(width) 4.3 inches		
Cuff size	22~42 cm (8.66~16.53 inches)		
Service life	5 years		
Protection against electric shock	Internally powered supply		
	Temperature	5°C~40°C	If stored or used

Operating environment	condition		beyond the designated temperature and humidity range, it will not be used properly
	Humidity condition	15%~90%RH	
	Atmospheric condition	70kPa~106kPa	
Transportation and Storage Environment	Avoid strong impact, direct impact, exposure or rain during transportation. The packaged Blood Pressure Monitor shall be stored indoors at the temperature of -20°C~55°C and the relative humidity of 10%~93% ,atmospheric Condition: 70kPa~106kPa. without corrosive gas and with good ventilation.		

17. Appendix 1 EMC Information

Guidance and manufacturer's declaration - Electromagnetic emission		
The Arm Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the Arm Blood Pressure Monitor should assure that it is used in such an environment.		
Emissions	Compliance	Electromagnetic environment - guidance
RF emissions CISPR 11	Group 1	The Arm Blood Pressure Monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

RF emissions CISPR 11	Class B	The Arm Blood Pressure Monitor is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC61000-3-2	N.A.	
Voltage fluctuations/flicker emissions IEC61000-3-3	N.A.	

Guidance and manufacturer's declaration - Electromagnetic immunity

The Arm Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of Arm Blood Pressure Monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level
Electrostatic discharge (ESD) IEC 61000-4-2	±8kV contact ±2 kV,±4kV,±8 kV, ±15kV air	±8 kV contact ±2 kV,±4 kV, ±8 kV,±15 kV air
Electrostatic discharge (ESD) IEC 61000-4-2	±8 kV contact ±2 kV,±4 kV, ±8 kV,±15 kV air	±8 kV contact ±2 kV,±4 kV,±8 kV, ±15 kV air
Electrical fast transient/burst IEC 61000-4-4	Not applicable	Not applicable
Surge IEC 61000-4-5	Not applicable	Not applicable

RESTA

Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	Not applicable	Not applicable
Power frequency Magnetic field IEC 61000-4-8	30A/m, 50/60Hz	30A/m, 50/60Hz
Conducted RF IEC61000-4-6	Not applicable	Not applicable
Radiated RF IEC61000-4-3	10 V/m 80 MHz - 2,7 GHz 80 % AM at 1 kHz	10 V/m 80 MHz - 2,7 GHz 80 % AM at 1 kHz
NOTE: UT is the a.c. mains voltage prior to application of the test level		

Guidance and manufacturer's declaration - electromagnetic Immunity

The Arm Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the Arm Blood Pressure Monitor should assure that it is used in such an environment.

Radiated RF IEC6100 0-4-3 (Test specifications for ENCLOSURE PORT IMMUNITY to RF wireless communications equipment)	Test Frequency (MHz)	Band (MHz)	Service	Modulation	Max. Power (W)	Distance (m)	IEC 60601-1-2 Test Level (V/m)	Compliance level (V/m)
	385	38 0-390	TETRA 400	Pulse modulation 18 Hz	1,8	0,3	27	27
	450	43 0-470	GMRS 460, FRS 460	FM \pm 5 kHz deviation 1 kHz sine	2	0,3	28	28

RESTA

710	70	LTE Band 13, 17	Pulse modula tion 217 Hz	0,2	0. 3	9	9
745	4-7						
780	87						
810	80	GSM 800/90 0, TETRA 800, iDEN 820, CDMA 850, LTE Band 5	Pulse modula tion 18 Hz	2	0. 3	28	28
870	0-9						
930	60						
1720	1	GSM 1800; CDMA 1900; GSM 1900; DECT; LTE Band 1, 3, 4, 25; UMTS	Pulse modula tion 217 Hz	2	0. 3	28	28
1845	70						
1970	0 – 1 99 0						
2450	2	Blueto oth, WLAN, 802.11 b/g/n, RFID 2450, LTE Band 7	Pulse modula tion 217 Hz	2	0. 3	28	28
	40						
	0 – 2 57 0						
5240	5	WLAN 802.11 a/n	Pulse modula tion 217 Hz	0,2	0. 3	9	9
5500	10						
5785	0 – 5 80 0						

Guidance and manufacturer's declaration - electromagnetic Immunity

Radiated RF IEC61000-4-39 (Test specifications for ENCLOSURE PORT IMMUNITY to proximity magnetic fields)	Test Frequency	Modulation	IEC 60601-1-2 Test Level (A/m)	Compliance level (A/m)
	30 kHz	CW	8	8
	134.2 kHz	Pulse modulation 2.1 kHz	65	65
	13.56 kHz	Pulse modulation 50 kHz	7.5	7.5

NOTE:

- Optional AC adaptor should comply with the requirement of IEC 60601-1 standard.
- Use only the exclusive AC adapter specified by authorized dealers. Other AC adapter may vary in output voltage and polarities and may represent a risk on your life and damaging the device.

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English

Our manuals are available in multiple languages and in larger format. You can find them online on our website: <https://www.cresta-care.com> Download the manual in your language now!

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English: Medical device

Svenska: Medicinteknisk produkt

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Shenzhen AOJ Medical Technology Co., LTD
Address: RM 301 4F Block A Building A Jingfa Intelligent
Xixiang Street, Bao'an District 518126 Shenzhen
People's Republic Of China

Designed in The Netherlands / Made in China

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CRESTA INTERNATIONAL B.V.